ISTRUZIONI D'USO

Art. AK100 - AK101

SUPPORTO LOMBARE

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Art. AK100 - AK101

LUMBAR SUPPORT

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

THERAPEUTIC INDICATIONS

Care an prevention of lumbar back pain, in all lumbar injuries

PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not raccomandated to wear the support near open flames

-It is raccomandated to replace the device when there is wear or tear, even partial

MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and sotre in a cool place.

APPLICATION INSTRUCTION

1) Front fastening. ajustement through the belt placed on the left side of the Orthosis. It's important that the brace is centred. It's possible to adjust the width of the support throught back panels and to increase the back push through straps positioned on the lumbar plate.

It's advisable to consult an Orthopaedic Technician or a Doctor for any ajustement.

NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We reccomend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misure of the product.