ISTRUZIONI D'USO

Art. CK200 - CK201

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USER GUIDE

Art. CK200 - CK201

SOFT CERVICAL COLLAR - WITH REINFORCEMENTS

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

THERAPEUTIC INDICATIONS

Reccomended in post-operative management and in general instabilities.

PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not raccomended to wear the support near open flames

-It is raccomended to replace the device when there is wear or tear, even partial

MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and sotre in a cool place.

APPLICATION INSTRUCTION

- 1) rest in chin in the apposite space; rear locking with no costrains.
- 2) Generally the collar has to keep your neck in a neutral position.
- 3) In model CK201 the support reinforcement can be removed under the indications of a Doctor or after the acute phase and the collar can be used as a simple soft collar.

NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We reccomend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misure of the product.