ISTRUZIONI D'USO

Art. CK202 - CK203

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Art. CK202 - CK203

RIGID CERVICAL COLLAR - WITH CHIN

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

THERAPEUTIC INDICATIONS

Cervical myalgia, immobilization after traumas

PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not recommended to wear the support near open flames

-It is recommended to replace the device when there is wear or tear, even partial

MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and store in a cool place.

APPLICATION INSTRUCTION

- 1) Rest the chin in the opposite space, rear locking with no constraints
- 2) Generally the collar has to keep your neck in a neutral position.

N.B. For a correct application and use of the brace contact the Orthopaedic Technician or a Doctor

NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application. We recommend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misuse of the product.