## ISTRUZIONI D'USO

### Art. GK560 - GK570

#### GINOCCHIERA POST OPERATORIA

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# Art. GK560 - GK570

### POST OPERATIVE KNEE BRACE

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

#### THERAPEUTIC INDICATIONS

Reccomended in post-operative period and rehabilitation

#### PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not raccomended to wear the support near open flames

-It is raccomended to replace the device when there is wear or tear, even partial

#### MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and sotre in a cool place.

#### APPLICATION INSTRUCTION

- 1)Put on the brac, centring the articular joints to the patella while the splints have to be parallel to the limb
- 2)Close panels at thigh and calf
- 3) Fasten the straps starting from the thigh to the calf and re-adjust them once the Orthosis adhere to the limb
- 4) Before standing up, its important to re position the brace 2/3 cm. Upwards adn re-adjust the velcro straps, if necessary.
- 5) To take off the brace follow the inverse sequence to points 2/3

N.B. For the first application it's important to consult an Orthopaedic technician. Who must decide the exact splits position and adjust flexo-extension (if indicated by a Doctor)

#### NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We reccomend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misure of the product.