ISTRUZIONI D'USO

Art. PK08

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USER GUIDE

Art. PK08

VOLAR WRIST BRACE FOR THE CONTROL THE 1° METACARPO-PHALENGEAL JOINT

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

THERAPEUTIC INDICATIONS

Recommended in post-operative management, in general instabilities and to control wrist movements

PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not recommended to wear the support near open flames

-It is recommended to replace the device when there is wear or tear, even partial

MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and store in a cool place.

APPLICATION INSTRUCTION

1) Put on the support, inserting the thumb in the specific pocket. The velcro strap near fingers has to be closed.

2) Close the other velcro straps progressively from the fingers to the wrist. Adjust again the first, nearest to fingers, velcro fastenings. The support has to be adherent to the limb.

4) Ensure the stability closing the velcro strap around the wrist.

N.B. Can be used for left and right. Before applying it, the Orthopaedic technician or the Doctor have to position and, if necessary, mold the volar splint and the splint at the first metacarpo-phalangeal joint. The orthosis is supplied ready for the right hand. verify that there is a residual elasticity to prevent stasis.

5) Notes for the orthopaedic technician: invert the position of the volar splint for the application on the left hand.

NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We recommend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misuse of the product.