## ISTRUZIONI D'USO

### Art. PK23

#### CINTURINO SOTTOROTULEO

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### Art. PK23

#### UNDER KNEE-CAP STRAP

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

#### THERAPEUTIC INDICATIONS

Advisable in case of patellar tendomyopathies

#### PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not recommended to wear the support near open flames

-It is recommended to replace the device when there is wear or tear, even partial

#### MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and store in a cool place.

#### APPLICATION INSTRUCTION

- 1) Position the pad under the kneecap
- 2) Adjust the band putting the part with the velcro trap on the pad
- 3) ambidexter

N.B. Verify that there is a residual elasticity to prevent stasis.

#### NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We recommend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misuse of the product.