ISTRUZIONI D'USO

Art. PK24 - PK25 - PK26

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USER GUIDE

Art. PK24 - PK25 - PK26

KNEE BRACE WITH PATELLAR PATCH - PATELLAR CONTROL - PATELLAR WITH TOTAL CONTROL

Thank you for choosing NewellMed synonym of quality and competence in orthopedic field. Please read the following instruction carefully for a correct use of the product.

THERAPEUTIC INDICATIONS

Advisable in case of shoulder instabilities, osteoarthritis, rheumatoid arthrosis and hemiplegia

PRECAUTIONS FOR USE

-Position the support without exerting too much pressure and avoiding the application on injured or excessive swelling.

-For any question, please contact the Orthopaedic Technician.

-It's not recommended to wear the support near open flames

-It is recommended to replace the device when there is wear or tear, even partial

MAINTENANCE

-hand washing with maximum temperature 30° using mild soap.

-Don't bleach.

-Don't iron.

-Dry away from direct heat sources and store in a cool place.

APPLICATION INSTRUCTION

- 1) Put on the knee brace folding the part with the non-slip silicon part and pull the brace with both hands.
- 2) The hole has to be placed at the centre of the patella
- 3) Insert the patellar semi-ring following therapeutic indications. For model PK26 adjust the control tie-rods on the semi-ring.
- 4) Take off the support folding again the silicon part.

N.B. Verify that there is a residual elasticity to prevent stasis.

NOTES AND WARNINGS

Class I medical device, manufactured by Contel S.r.l. at its headquarters in Strada delle Brigole, 46 23877 Paderno d'Adda (LC) Italy in accordance with Directive 93/42 CE.

For the use of this device is advisable to consult a doctor or orthopedic technician to avoid any contraindications.

The proper functioning of the brace is the result of a correct application We recommend personal use of the product.

In case of malfunction contact your local dealer or contact Contel S.r.l.. The company is not liable for misuse of the product.